

glide through  
quickly and  
easily



## pro • glider™

The ONE Glide Path File



+  
WE  
KNOW  
ENDO.

- Suitable for highly curved cases
- Reduced canal transportation
- Simple to use

For better dentistry

**DENTSPLY**  
MAILLEFER

# pro•glider™


One file with several benefits

Suitable for most root canals, including highly curved ones


Tip size 16.02 equipped with a variable progressive taper in combination with the M-Wire™ nickel titanium alloy.




- Improves file flexibility, while still retaining cutting efficiency
- Provides greater resistance to cyclic fatigue, the leading cause of file separation

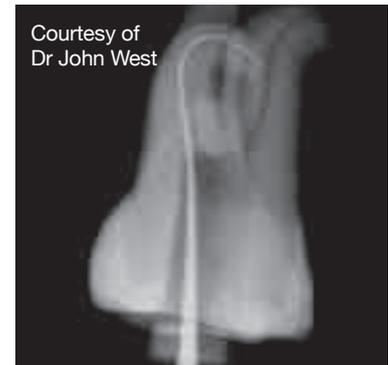

## Short Sequence

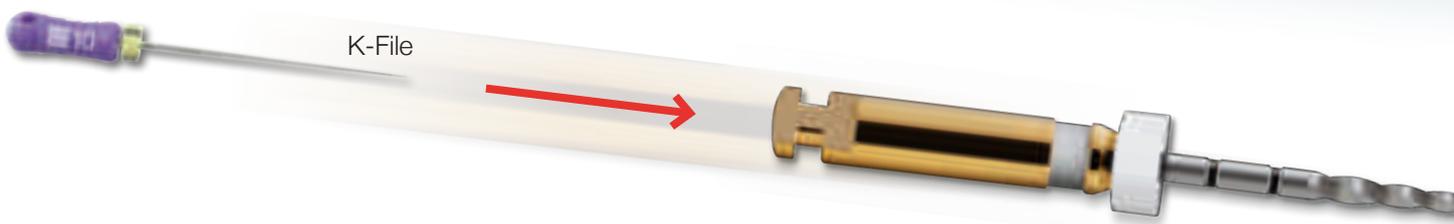

Less shaping time provides more time for irrigation

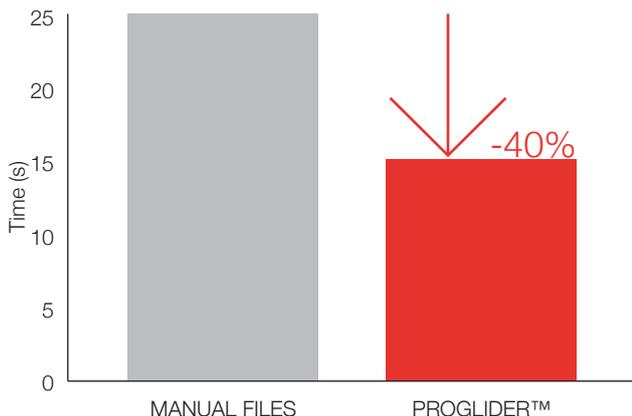

- Simple to use
- One single mechanized instrument
- Less files – less confusion – more safety

## Respect of the root canal anatomy



□ Berutti et al. 2013



Conventional stainless steel file – high aberration prevalence (zips, ledges and transportation)



PROGLIDER™ – significantly less aberrations

**pro•glider™**

Can be used with commercially available motor systems.

**Motor setting:**  
Speed: 300 rpm  
Torque: 2-5.2 Ncm

wave•one<sup>®</sup>  
GOLD

efficiency and simplicity  
in perfect harmony

PROGLIDER™ enhances the simplicity of the WAVEONE® GOLD solution by making it possible to obtain a glide path with only one mechanized instrument prior to one single WAVEONE® GOLD shaping file.

**PROTAPER•NEXT™**

easy glide meets  
flexible performance

PROGLIDER™ is suitable for most root canals, including the highly curved ones, whilst respecting the root canal anatomy. It's therefore an excellent choice combined with PROTAPER NEXT™.

## Single patient use




Pre-sterilised blister packs for single patient use

- Optimal performance every time
- Improved Economic Efficiency

No risk for cross contamination

- Increased safety for you and your patient

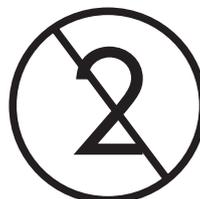

# A Complete Solution

NiTi FILES



## PROGLIDER™ File

Single Size: 016/.02  
 A092622100103 PROGLIDER File Sterile 21mm – blister of 6  
 A092622500103 PROGLIDER File Sterile 25mm – blister of 6  
 A092623100103 PROGLIDER File Sterile 31mm – blister of 6



wave•one<sup>®</sup>  
GOLD



Choose your endo solution

PROTAPER•NEXT™



## WAVEONE® GOLD Files



### small reciprocating file

6x small (020 07) – 21mm – A 0756 221 G0S 03  
 6x small (020 07) – 25mm – A 0756 225 G0S 03  
 6x small (020 07) – 31mm – A 0756 231 G0S 03



### primary reciprocating file

6x small (025 07) – 21mm – A 0756 221 G0P 03  
 6x small (025 07) – 25mm – A 0756 225 G0P 03  
 6x small (025 07) – 31mm – A 0756 231 G0P 03



### medium reciprocating file

6x small (035 06) – 21mm – A 0756 221 G0M 03  
 6x small (035 06) – 25mm – A 0756 225 G0M 03  
 6x small (035 06) – 31mm – A 0756 231 G0M 03



### large reciprocating file

6x small (045 05) – 21mm – A 0756 221 G0L 03  
 6x small (045 05) – 25mm – A 0756 225 G0L 03  
 6x small (045 05) – 31mm – A 0756 231 G0L 03



### Assortment reciprocating file

1x small/primary/medium/large – 21mm  
 A 0754 221 G9A 03



1x small/primary/medium/large – 25mm  
 A 0754 225 G9A 03



1x small/primary/medium/large – 31mm  
 A 0754 231 G9A 03



## WAVEONE® GOLD Paper points



Box of 180x sterile paper points

small – A 1750W 000 00S 03

primary – A 175W 000 00P 03

medium – A 175W 000 00M 03

large – A 175W 000 00L 03



## WAVEONE® GOLD Gutta percha points



Box of 60x variable

tapered gutta percha points

small – A 1750X 000 00S 00

primary – A 175X 000 00P 00

medium – A 175X 000 00M 00

large – A 175W 000 00L 00



## PROTAPER NEXT NiTi FILES



REF A080022100103	X1	File Ster. 21mm	017 .04	6x
REF A080022500103	X1	File Ster. 25mm	017 .04	6x
REF A080023100103	X1	File Ster. 31mm	017 .04	6x



REF A080022100203	X2	File Ster. 21mm	025 .06	6x
REF A080022500203	X2	File Ster. 25mm	025 .06	6x
REF A080023100203	X2	File Ster. 31mm	025 .06	6x



REF A080022100303	X3	File Ster. 21mm	030 .07	6x
REF A080022500303	X3	File Ster. 25mm	030 .07	6x
REF A080023100303	X3	File Ster. 31mm	030 .07	6x



REF A080022100403	X4	File Ster. 21mm	040 .06	6x
REF A080022500403	X4	File Ster. 25mm	040 .06	6x
REF A080023100403	X4	File Ster. 31mm	040 .06	6x



REF A080022100503	X5	File Ster. 21mm	050 .06	6x
REF A080022500503	X5	File Ster. 25mm	050 .06	6x
REF A080023100503	X5	File Ster. 31mm	050 .06	6x



### Assortment X1 – X3

REF A08002219A003	X1 – X3	Ass.X1-X3	Ster.21	6x
REF A08002259A003	X1 – X3	Ass.X1-X3	Ster.25	6x
REF A08002319A003	X1 – X3	Ass.X1-X3	Ster.31	6x

## PROTAPER Paper points



REF A022Q00000200	X2	sterile paper points	180x
-------------------	----	----------------------	------



REF A022Q00000300	X3	sterile paper points	180x
-------------------	----	----------------------	------



REF A022Q000090000	X2	sterile paper points	90x
--------------------	----	----------------------	-----



	X3	sterile paper points	90x
--	----	----------------------	-----



REF A022Q000090100	X4	sterile paper points	90x
--------------------	----	----------------------	-----



	X5	sterile paper points	90x
--	----	----------------------	-----

## PROTAPER Gutta percha points



REF A022000000200	X2	gutta percha points	60x
-------------------	----	---------------------	-----



REF A022000000300	X3	gutta percha points	60x
-------------------	----	---------------------	-----



REF A0220000090000	X2	gutta percha points	30x
--------------------	----	---------------------	-----



	X3	gutta percha points	30x
--	----	---------------------	-----



REF A0220000090100	X4	gutta percha points	30x
--------------------	----	---------------------	-----



	X5	gutta percha points	30x
--	----	---------------------	-----

**DENTSPLY Nordic Baltic**  
 Box 2024, SE-128 21 Skarpnäck  
 Tel +46 (0)8 685 65 05  
 Fax +46 (0)8 447 60 55  
 www.dentsply.eu

**For local contact, please visit**  
[www.dentsply.se](http://www.dentsply.se)  
[www.dentsply.fi](http://www.dentsply.fi)  
[www.dentsply.no](http://www.dentsply.no)  
[www.dentsply.dk](http://www.dentsply.dk)  
 Tel LV + 371 29 455 945  
 Tel LT +370 60 580 858

**For more information, please visit**  
[www.dentsplymallefer.com](http://www.dentsplymallefer.com)



0459

For better dentistry

**DENTSPLY**  
**MAILLEFER**