


Cleanic[®]

The one step prophylaxis paste



ALL IN ONE

Universal prophylaxis paste
Guarantees outstanding
Results

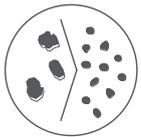

PATENTED PERLITE TECHNOLOGY

Featuring integrated
Abrasion variability



SCIENTIFICALLY PROVEN

Effective cleaning & polishing with
Low abrasion of Enamel and dentine



TIME SAVING

From cleaning to polishing
in one step



Cleanic is the one step prophylaxis paste that guarantees your patient outstanding cleaning & polishing results with low abrasion of enamel and dentine in a one-step procedure.

That's because of the clinically proven integrated abrasion variability of natural perlite particles



WHERE TRADITIONAL PROPHY PASTES FALL SHORT...

Traditional pastes, usually based on pumice, are aggressive towards enamel, dentine and fillings, causing:

DISCOLOURATION
AND FORMATION
OF CALCULUS



SCRATCHES
TO ENAMEL
AND DENTINE



UNNECESSARY
DAMAGE
TO HARD TISSUE



PALPABLE LOSS
OF TISSUE
OVER TIME



They also require twice the work, needing at least two different grit pastes to be effective. This often results in polishing being omitted from the procedure in an effort to save time, with the remaining unpolished surface becoming discoloured over time and accumulating new plaque.

WHAT'S DIFFERENT WITH CLEANIC?

Cleanic is a **one-step cleaning and polishing prophy paste** based on perlite technology.

This means gentle but efficient cleaning and polishing, all done in one session, with one paste.



Cleanic available in 5 tastes



Berry Burst



Mint



Green Apple



Light



Mint Fluoride Free

INTEGRATED ABRASION VARIABILITY


The volcanic Perlite (natural silica) particles adapt their abrasiveness throughout your procedure.

CLEANING

When Cleanic is first applied to the tooth surface, it behaves like a coarse paste. This is when Cleanic's cleaning ability is in full force.


POLISHING

Under load and within seconds, the abrasive particles are fractionated and their cutting edges become rounded and less blunt. At this point, the paste becomes effective for a high gloss polishing.

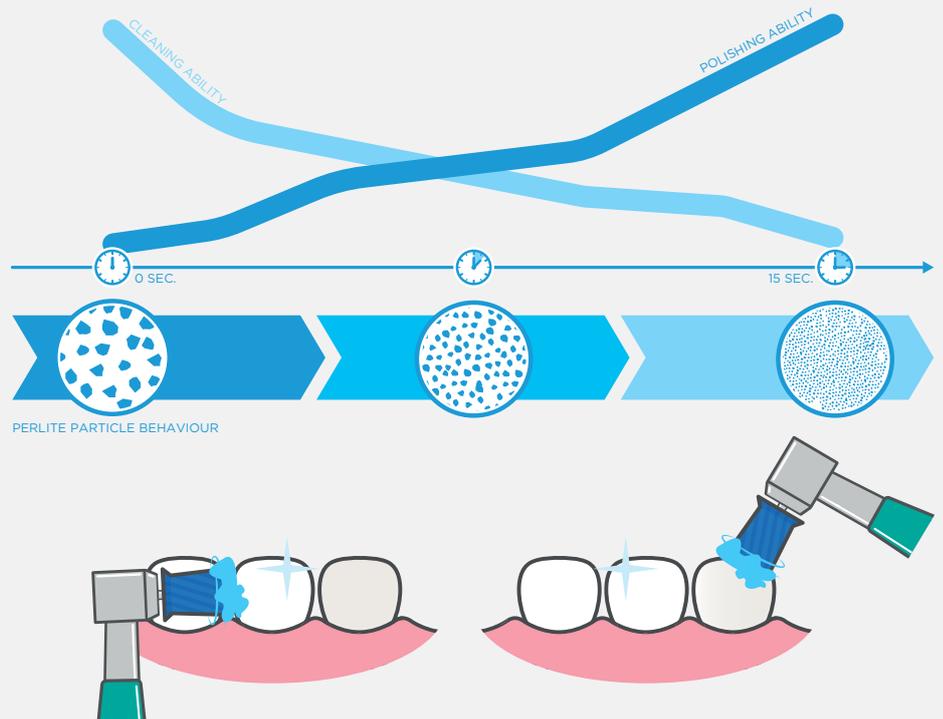

HOW TO USE CLEANIC

Apply the Cleanic paste to a limited group of teeth at a time (about 3 teeth) using a rubber cup (i.e. Pro-Cups) to work it from cleaning to polishing, all in one step. Once done, apply a fresh portion of Cleanic paste to treat the next group of teeth.

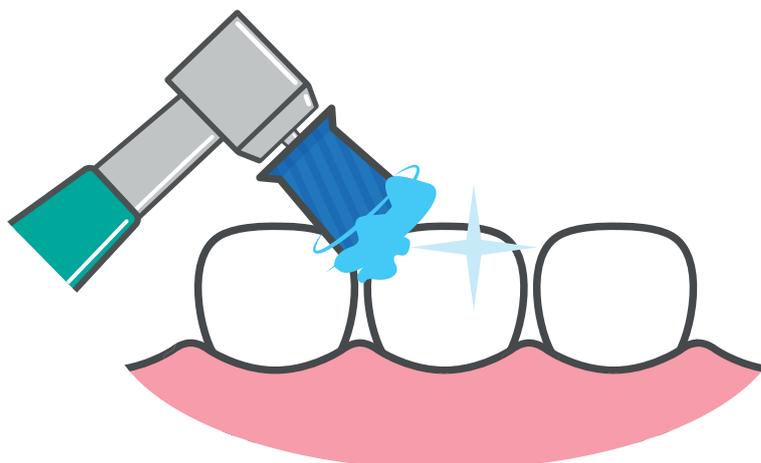

FOR NORMAL DENTITION:

Work from the incisal/occlusal surface of each tooth, moving towards the gingival area; this will minimise abrasion on the tooth.

FOR EXPOSED DENTINE OR ROOT SURFACES:

Start by applying a fresh portion of paste to the enamel surface. Move towards the exposed root surfaces or dentine with the used paste (so it is less aggressive).

ORDER YOUR CLEANIC NOW!

Part No.	Description	Content	Fluoride	RDA	REA	Taste	Menthol	Paste Colour
3110	Cleanic Cartridge Refill Packs	200g	0.15% CaF	27	3.4	Mint	Y	Light blue
3130	Cleanic Cartridge Refill Packs	100g	0.15% CaF	27	3.4	Mint	Y	Light blue
3210	Cleanic Cartridge Refill Packs - Fluoride free	200g	-	27	3.4	Mint	Y	Light blue
3230	Cleanic Cartridge Refill Packs - Fluoride free	100g	-	27	3.4	Mint	Y	Light blue
3140	Cleanic Single Dose	200x 2g	0.10% NaF	27	3.4	Mint	Y	Dark blue
3180	Cleanic in Tube Mint	100g	0.10% NaF	27	3.4	Mint	Y	Blue
3182	Cleanic in Tube Green Apple	100g	0.10% NaF	27	3.4	Green Apple	N	Green
3183	Cleanic in Tube Mint Fluoride Free	100g	-	27	3.4	Mint	Y	Light blue
3184	Cleanic in Tube Light	100g	0.10% NaF	27	3.4	--	N	Light grey
3186	Cleanic in Tube Berry Burst	100g	0.10% NaF	27	3.4	Red fruits	N	Pink
3190	Cleanic Collection Kit	3x 100g Tube Cleanic Prophy Paste (Mint with Fluoride, Green Apple, Light), Tube Squeezer, Reusable Bag.						

TRY CLEANIC WITH ITS IDEAL PARTNERS!

Pro-Cup and Pro-Cup Junior

The laminated profile of these single-use cups helps avoid the formation of splash balls while applying the prophy paste. The same profile also helps mitigate overheating. Pro-Cup and Pro-Cup Junior are not made of natural latex.

	CODES & QUANTITIES				COLOUR	DESCRIPTION	TYPE		SINGLE USE
	CODE	QTY	CODE	QTY			LATCH	SCREW	
	990/30	30	990/120	120	●	Pro-Cup - Soft	✓		✓
	991/30	30	991/120	120	●	Pro-Cup - Hard	✓		✓
	1810/30	30	1810/120	120	●	Pro-Cup - Soft		✓	✓
	1811/30	30	1811/120	120	●	Pro-Cup - Hard		✓	✓
	992/30	30	992/120	120	●	Pro-Cup Junior - Soft	✓		✓
	993/30	30	993/120	120	●	Pro-Cup Junior - Hard	✓		✓
	1812/30	30	1812/120	120	●	Pro-Cup Junior - Soft		✓	✓
	1813/30	30			●	Pro-Cup Junior - Hard		✓	✓



Kerr's paste carriers

Available as single use or multiple use, these ergonomic prophy paste carriers are easy to fit to your finger, and have enough space/volume even for bigger amounts of paste (when/if needed).

Ring Pots - Not autoclavable

Part No.	Description
985	Ring Pots – Blue – Pack of 100
986	Ring Pots – Assorted Colours – Pack of 100

Prophy Clips - Autoclavable

Part No.	Description
3151	Prophy Clips – Pack of 6