



Manual applicator

PRESENTATION

Box containing: 1 manual applicator,

CHARACTERISTICS

- Easily removed capsule-holder.
- Applicator and capsule-holder sterilisable.

INDICATIONS

The EXPASYL™ applicator is used for positioning the EXPASYL™ paste in the sulcus.

INSTRUCTIONS FOR USE

CAUTION: It is recommended to disinfect the applicator before its first use.

1 - Installation of the capsule:

• Insert the cannula in the capsule. • Pull the applicator piston back using the button while grasping the applicator body. • Insert the capsule fitted with its cannula into the capsule-holder. • Advance the piston towards the capsule by pressing the applicator trigger several times until the paste begins to pump out.

2 - Capsule replacing/Removal:

• Pull the applicator piston back using the button while grasping the applicator body. • Remove the cannula-capsule assembly and insert a new capsule, if necessary, as indicated below.

DISASSEMBLING OF CAPSULE-HOLDER

Turn the capsule-holder through approximately one 8th of a turn and pull on it (bayonet system).

MAINTENANCE

1 - Cleaning the capsule-holder:

Clean and decontaminate using usual disinfecting solutions (with or without aldehyde) respecting the conditions of dilution and use given by the manufacturer; rinse thoroughly.

2 - Cleaning the applicator body:

Clean and decontaminate using disinfecting wipes (without aldehyde) respecting the conditions of use given by the manufacturer.

CAUTION: Never immerse the applicator in disinfecting solutions. Never lubricate the applicator.

3 - Sterilization:

Conditions: Autoclave 134 °C - 2 bars - 4 min

Note: It is important to remove any residual paste on the applicator after each use and prior to sterilisation to prevent corrosion of the applicator by the paste.

DISPOSAL

Recycle or dispose of according to current legislations, rather by a waste collector or an approved company. Do not dispose of in the environment.



Professional dental product.

Classification according to European Directive 93/42/EEC

Medical device class I

PRODUITS DENTAIREs PIERRE ROLLAND - 17 avenue Gustave Eiffel - 33700 MERIGNAC- France - Tel. : +33 (0) 556 34 06 07