

Tubulex[®] – PROTECTION AFTER TOOTH EXTRACTION

Tubulex is used after tooth extraction to accelerate the healing process and prevent the emergence of “Dry Socket” syndrome. Tubulex is pre-packaged in air-tight single portions and moistened with an antibacterial solution creating an antiseptic zone of approx. 30-40 mm around the cavity.

Easy to use:


1. Open the aluminium foil plate by pressing with the thumb.
2. Take out a dressing using tweezers.
3. Fold the dressing into a “V” shape before applying.
4. Apply 1-2 dressings to cover the alveolar.

Facts and benefits

- **Antibacterial solution** - Minimizes pain and inflammatory symptoms. Creates an antiseptic zone of approx. 30–40 mm around the cavity
- **Multi-purpose treatment** - Hastens the healing process after extraction. Prevents the emergence of “Dry Socket” syndrome
- **Easy to use** - Disposable sterile dressing. Pre-packaged in air-tight single portions



Did you know:
Tubulex hastens the healing process after tooth extraction preventing the emergence of “Dry Socket” syndrome.

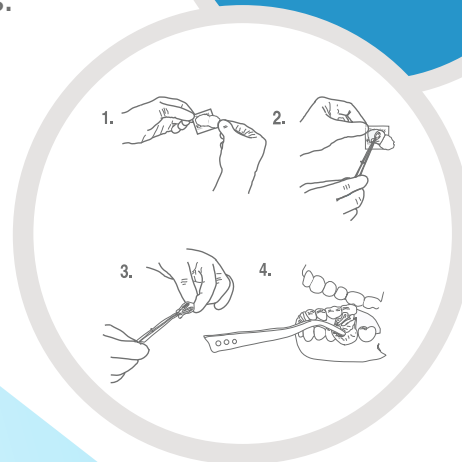

Dental Therapeutics

phone: +46(0)8 716 81 41

email: info@therapeutics.se

web: www.therapeutics.se